

AMXDmax

ADVANCED MISSION EXTENDER DEVICE

Pad w/Removable Hose
*Introductory
Manual*



Omni Medical Systems, Inc.
AMXDmax.com

Pad w/Removable Hose

Pad w/Removable Hose

Inflatable urethane pad design based on a sanitary pad. Designed for optimal use in all aircraft, including aircraft with the 5-point harness.

Removable Hose Standard Issue or TSC Compatible

The removable hose connects to the urine sensors in the pad, attaching to the pad's front.

Control Unit and Bag

The Pad works with the AMXDmax control unit and collection bag.




Attach Hose

Attach the hose to the white connector. Test the hose connection by pulling on the hose to make sure it doesn't detach.



Insert Pad

Place the pad into the garment at the correct position. See the manual for correct pad positioning

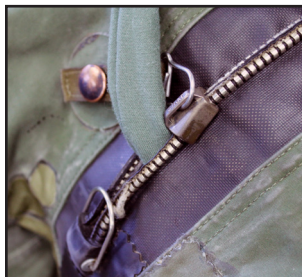

Connect to System

Leave hose accessible via garment zipper or other access point. Connect to AMXDmax Control Unit and Bag for use.

TSC Compatible Garments



Connect to TSC

If using the pad with a TSC enabled garment, attach the domed end of the **TSC Compatible Hose** to the inside (white side) of the TSC port.



Connect TSC to System

Connect the TSC Hose to the external TSC port, and then to the AMXDmax Control Unit and Bag for use.



Through Suit Connector (TSC)

When used with a TSC enabled garment, the device can easily connect and disconnect from the suit, without the need to unzip, providing the ultimate in discrete operation.

Environmentally Sealed

With the Through Suit Connector (TSC), the system is fully self-contained and environmentally sealed, perfect for CBRN ensembles and immersion suits.



Fully Automatic Self Contained Aircrew Wearable



Through Suit Connector Kit

The TSC Kit allows for easy modification of any garment to enable use with TSC compatible equipment.



Omni Medical Systems, Inc.

808 Hercules Drive, Colchester, VT 05446

☎: (802)497-2253 📠: (802)497-3601 www.amxdmax.com