

AMXDmax

ADVANCED MISSION EXTENDER DEVICE

Cup w/Removable Hose

Introductory Manual



Omni Medical Systems, Inc.
AMXDmax.com

Cup w/Removable Hose

Cup w/Removable Hose

Flexible urethane cup. Designed for optimal use in all aircraft, including aircraft with the 5-point harness.

Removable Hose

Standard Issue or TSC Compatible

The removable hose contains the urine sensors and connects to the bottom of the CUP.

Control Unit and Bag

The Cup works with the AMXDmax control unit and collection bag.



Attach Hose

Attach the black cup to the hose connector. Ensure that the hose snaps securely in place. Rotate hose for desired location.



Insert Cup

Place the cup into the garment pocket and insert the male anatomy through the foam ring. Secure the pocket and position the hose as desired.



Connect to System

Leave hose accessible via garment zipper or other access point. Connect to AMXDmax Control Unit and Bag for use.



Alignment Flaps

If necessary, the alignment flaps can be cut off and discarded to provide a more comfortable fit.

Depending on which provides a better fit, future supplies can be ordered with or without flaps as desired.

TSC Compatible Garments



Connect to TSC

If using the cup with a TSC enabled garment, attach the White end of the **TSC Compatible Hose** to the inside (white side) of the TSC port.



Connect TSC to System

Connect the TSC Hose to the external TSC port, and then to the AMXDmax Control Unit and Bag for use.



Through Suit Connector (TSC)

When used with a TSC enabled garment, the device can easily connect and disconnect from the suit, without the need to unzip, providing the ultimate in discrete operation.

Environmentally Sealed

With the Through Suit Connector (TSC), the system is fully self-contained and environmentally sealed, perfect for CBRN ensembles and immersion suits.



Fully Automatic Self Contained Aircrew Wearable



Through Suit Connector Kit

The TSC Kit allows for easy modification of any garment to enable use with TSC compatible equipment.



Omni Medical Systems, Inc.

808 Hercules Drive, Colchester, VT 05446

☎: (802)497-2253 📠: (802)497-3601 www.amxdmax.com